


Designed for unexpected heroes™

ZOLL AED 3®

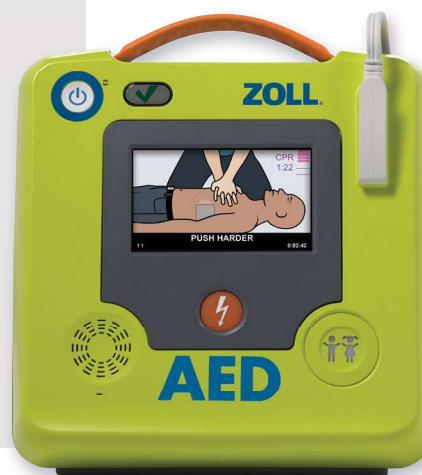

Enhanced Real CPR Help

The ZOLL AED 3® automated external defibrillator (AED) provides detailed CPR guidance with proprietary Real CPR Help® technology. More powerful full-colour images and a CPR cycle timer, combined with voice and text prompts, help guide responders through the rescue. The Real CPR Help integrated, real-time CPR feedback tells and shows rescuers when they need to “Push Harder” or whether they are doing “Good Compressions,” so they know when they are administering high-quality CPR.

RapidShock® Analysis

The ZOLL AED 3 with RapidShock® analysis enables the shortest rhythm analysis for more continuous care and more lifesaving CPR. RapidShock analysis allows the ZOLL AED 3 to deliver a shock, if needed, in as little as 5 seconds. Minimising the pre-shock pause and providing more CPR can improve patient outcomes.*

Research shows that minimising time to shock after the CPR cycle ends may improve survival.¹ The ERC Guidelines note, “The delay between stopping chest compressions and delivery of the shock (the pre-shock pause) must be kept to an absolute minimum; any delay will reduce the chances of the shock being successful.”²

Integrated Paediatric Rescue

Paediatric rescue is made easier with ZOLL’s unique CPR Uni-padz® electrodes and a Child Mode setting. Universal pad design provides rescuers a single solution to treat both adult and paediatric victims of SCA by using the same set of electrode pads and simply activating Child Mode.

Low Total Cost of Ownership

Longer-lasting 5-year batteries and electrode pads reduce AED device maintenance requirements. Less maintenance can improve device readiness and has the added benefit of reducing the total cost of ownership over the life of your ZOLL AED 3 device.

Optional WiFi Connectivity

Easily manage your suite of devices via the cloud. ZOLL AED 3 devices are equipped with optional WiFi connectivity allow seamless communication with AED programme management systems through your wireless network. Once enabled, the ZOLL AED 3 will automatically report its readiness status, report a failed self-test, and send email alerts so you can take the appropriate corrective action and make sure your AEDs are ready in an emergency.


* On the ZOLL AED 3 Automatic, the pre-shock pause time is 8 seconds due to warning to the shock countdown alert before shock delivery.




Real CPR Help assists rescuers in delivering high-quality CPR by providing real-time feedback on chest compression rate and depth.



Activate Child Mode for pediatric rescue with CPR Uni-padz.

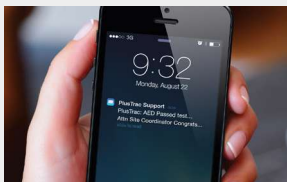

Optional cloud connectivity can enable automatic reporting of device status.

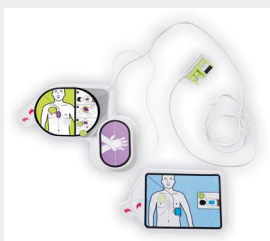

Universal CPR Uni-padz can be used for both adult and child rescues. They are the only pads that last 5 years and also include an integrated rescue kit.

ZOLL AED 3 Specifications

Defibrillator

Protocol: Semi-automatic and Fully Automatic configuration

Waveform: ZOLL Rectilinear Biphasic™

Defibrillator Charge Hold Time: 30 seconds; ZOLL AED 3 Automatic model: 5 seconds prior to automatic shock delivery

Energy Selection: Factory preprogrammed selection (Adult: 120 J, 150 J, 200 J; Child: 50 J, 70 J, 85 J)

Patient Safety: All patient connections are electrically isolated

Charge Time: Less than 10 seconds with new battery

Pre-shock Pause: 5 seconds with new battery (semi-automatic); 8 seconds with new battery (fully automatic)

Electrodes: ZOLL CPR Uni-padz™

Self-test: Configurable automatic self-test every day or every 7 days. Default: every 7 days. Monthly full-energy test (200 J).

Automatic Self-test Checks: Battery capacity, status, and expiration; electrode connection and expiration; ECG and charge/discharge circuits; microprocessor hardware and software; CPR circuitry and pads sensor; audio circuitry

Compression Depth: 1.9 cm to 10.2 cm; 0.75 in to 4 in

Defibrillation Advisory: Evaluates defibrillation pad connection and patient ECG to determine if defibrillation is required

Shockable Rhythms: Ventricular fibrillation with average amplitude >100 microvolts and wide complex ventricular tachycardia with rates greater than 150 BPM for adults, 200 BPM for pediatrics. For ECG analysis algorithm sensitivity and specificity, refer to ZOLL AED 3 Administrator's Guide.

Patient Impedance Measurement Range: 10 ohms to 300 ohms

Defibrillator: Protected ECG circuitry

Display Format: High-resolution LCD with capacitive touch panel

Display Screen Size: 5.39 cm x 9.5 cm; 2.12 in x 3.74 in

Data Recording and Storage:

User-configurable for 1 or 2 clinical events for total of 120 minutes. Includes ECG, impedance measurements, device prompts and CPR data.

Data Recovery: Controlled by touchscreen, uploaded to USB memory stick, or ZOLL CaseReview over a Wi-Fi network

Internal Clock Synchronization:

Coordinated Universal Time (UTC) synchronization occurs when communicating with the ZOLL Online server.

Device

Size: (H x W x D) 12.7 cm x 23.6 cm x 24.7 cm; 5.0 in x 9.3 in x 9.7 in

Weight: 2.5 kg; 5.5 lbs (with battery)

Power: Battery

Wireless: 802.11 a/b/g/n

Security Protocols: WPA 1, WPA 2, WPA Personal, WPA Enterprise

Port: USB 2.0

Rescue Accessory Pack Contents: Non-latex gloves, CPR barrier mask, razor, paper towel, and towelette

Medical Electrical Equipment:

Class II and internally powered by EN60601-I

Design Standards: Meets applicable requirements of EN 60601-1, IEC 60601-1-11, IEC 60601-2-4

MDR certified: approved under the European Union (EU) Medical Device Regulation 2017/745

PMA-approved: Holds pre-market approval under the US FDA

Environmental

Operating Temperature: 0° to 50°C; 32° to 122°F

Storage Temperature: -30° to 70°C; -22° to 158°F

Humidity: 10% to 95% relative humidity, non-condensing

Vibration: IEC 60068-2-64, Random, Spectrum A.4, Table A.8, Cat 3b; RTCA/DO-160G, Fixed Wing Aircraft, Section 8.6, Test Cat. H, Aircraft Zone 1

and 2, EN 1789, Sweep per EN 60068-2-6 Test Fc.

Shock: IEC 60068-2-27; 100G

Altitude: -381 m to 4573 m; -1250 to 15,000 ft

Particle and Water Ingress: IP55

Drop Test: 1 m; 3.28 ft

Battery

Battery Capacity: Typical new battery running at an ambient temperature of +20° C to +25° C (68° F to 77° F) can provide 140 defibrillator discharges at maximum energy (200 joules), or 6 hours of continuous monitoring (with 2-minute CPR periods). Note: CPR periods shorter than 2 minutes can decrease the operating time that can be obtained from a new battery.

Type: Disposable, sealed lithium manganese dioxide

Battery Standby Life (once installed): 5 years with weekly self-test. End of life indicated by blank status window (typical remaining shocks: 9).

Battery Shelf Life: Store for up to 2 years at 23°C (77°F) prior to installing in AED 3 to maintain above battery life

Temperature: 0°C to 50°C (32°F to 122°F)

Humidity: 10% to 95% (non-condensing)

Weight: 317.5 grams; 0.7 lbs

Size: (H x W x D) 27.75 mm x 133 mm x 88 mm; 1.0 in x 5.16 in x 3.5 in

Nominal Voltage: 12 volts

CPR Uni-padz

Shelf Life: 5 years

Conductive Gel: Polymer Hydrogel

Conductive Element: Tin

Packaging: Multilayer foil laminate pouch

Impedance Class: Low

Cable Length: 142 (+/-3.8) cm; 56 (+/- 1.5) in

Design Standards: Meets requirements of IEC 60601-2-4, 2010

¹² European Resuscitation Council website, <https://cprguidelines.eu/guidelines-2021>

ZOLL Medical Corporation

269 Mill Road | Chelmsford, MA 01824 | 978-421-9655 | 800-804-4356 | zoll.com

Specifications subject to change without notice.

©2025 ZOLL Medical Corporation. All rights reserved. RapidShock, Real CPR Help, Uni-padz, ZOLL, ZOLL AED 3, and ZOLL Rectilinear Biphasic are trademarks or registered trademarks of ZOLL Medical Corporation in the United States and/or other countries.

All other trademarks are the property of their respective owners.

Printed in U.S.A. MCN IP 2411 0963 -05 ERC

ZOLL®

For subsidiary addresses and other global locations, please go to www.zoll.com/contacts.