


INSTALLATION MANUAL FOR ROTAID SOLID PLUS

Determine the location where you wish to mount the Rotaid. Make sure that no irregularities are present at the desired spot on the wall. The preferred height of the cabinet is 1.60 meters (center of cabinet). Please take the fire safety requirements at the specific locations into account.

1. Use the mold to determine the position of the anchors in the wall. Drill all 4 holes and position the anchors.

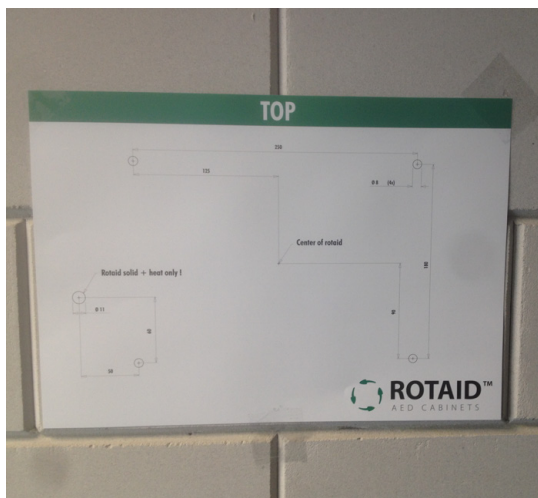

2. Attach the back plate to the wall via the 4 long screws and rings;



3. Use an Allen key (5 mm) to attach the nylon belt to the back plate. Make sure the belt is firmly attached.



4. After attaching the belt, please use both bolts to find the perfect holding position for your AED device. Make sure the bolts are firmly attached into the back plate adaptor.



5. You can now rotate the front cover on the back plate. Place the hood slightly counter-clockwise on the back wall and turn clockwise. When correctly placed, the hook on the bottom of the cabinet falls through the slot on the bottom of the cover.



6. Your Rotaid Solid Plus Cabinet is ready for usage!

