

samaritan[®] PAD 360P Trainer

Provides Realistic Training Demonstration

The easy-to-use samaritan PAD Trainer with remote control guides users through simulated analysis, energy delivery and prompted Cardio Pulmonary Resuscitation (CPR) with the look and feel of a live samaritan Public Access Defibrillator (PAD) 360P without the actual charge and discharge of an electrical shock.

Simulate the Full Range of "Real Life" Scenarios

The multi-function remote control enables you to select among six AHA/ERC scenarios for live demonstrations or training. One remote control can operate multiple trainer units at a time (within a range of 3-5 metres/10-15 feet). The Trainer is also compatible with any CPR manikin system.

Realistic

Voice and visual prompts and CPR coaching match those of the HeartSine samaritan PAD defibrillator. Likewise, size and shape are the same. Unique labeling and membrane color prevent inadvertent use in an actual emergency.

Appropriate for All Skill Levels

The Trainer can be used by businesses to train employee first responders on the use of the HeartSine samaritan PAD. In addition, it meets the needs of professional trainers, either teaching first-time lay users or refreshing experienced emergency responders.

Cost-Effective

The Trainer is powered by a rechargeable battery. Electrode pads are cost-effective and reusable, and mimic the Pad-Pak[™] battery/pad cartridge unique to the HeartSine samaritan PAD defibrillator.





The Whole Package. The HeartSine samaritan PAD Trainer package includes remote control with AAA batteries, Trainer-Pak, battery charger, soft carry case with two zipper compartments, user manual, data cable and five pairs of training electrode pads.

Scenarios

The HeartSine samaritan PAD Trainer integrates realistic rescue scenarios for AED training and adheres to the AHA/ERC 2010 protocol of one shock followed by two minutes of CPR. A metronome and flashing prompt at 110 beats per minute help to establish proper rate for chest compressions.

The selectable training scenarios include:

- SHOCK ADVISED (SA) condition, followed by CPR
- NO SHOCK ADVISED (NSA) condition, followed by CPR
- CHECK PADS, followed by an SA condition and CPR
- Starts directly in "ANALYZING" mode, followed by SA & CPR, SA & CPR, and NSA & CPR
- SHOCK ADVISED (SA) condition, followed by CPR, 30 compressions and 2 breaths
- NO SHOCK ADVISED (NSA) condition, followed by CPR, 30 compressions and 2 breaths



Product Features and Details

Features

Cable rewind mechanism. Easy replacement of pads and cables after each session reduces time spent resetting system.

Rechargeable Battery. Operates up to seven hours-a full day of training-on a single charge. Recharges in just six hours.

Battery status indicator. LEDs indicate charging status (while not in use) or one hour of remaining battery life (while in use).

Pause Button. Temporarily stops, then resumes, the training scenario to enable the trainer to offer additional explanation and support to the class.

Volume Configurable. Four selectable volume levels with default to last selection.

Selectable Training Scenarios. Six pre-programmed AHA/ERC scenarios.

Training Pads. Five pairs of reusable, replaceable training electrode pads included. Compatible with all training manikins, the new electrode pads are now easier to place back on the pad liner.

Upgradable. Updates to Trainer software available for download on the HeartSine website.

Language Options. Twenty-nine language options enable the Trainer to provide instructions in the local language.

Product Details (TRN-360-xx)

Specifications

- Weight: 0.6 kgs (1.3 lbs)
- Battery: Rechargeable
- Includes: Remote control with AAA batteries, Trainer-Pak, battery charger, soft carry case with two zipper compartments, user manual, data cable and five pairs of training electrode pads

Replacement Accessories

- Remote replacement (TRN-ACC-18)
- Data cable (PAD-ACC-02)
- Trainer-Pak electrode cartridge (TRN-PAK-04)
- Battery charger (TRN-ACC-14)
- Electrodes: 10/pak (TRN-ACC-02), 25/pak (TRN-ACC-03)

Lifesaving, Pure and Simple



Authorised Distributor

U.S./Americas

121 Friends Lane, Suite 400 Newtown, PA 18940 Toll Free: (866) 478 7463 Tel: + 1 215 860 8100 Fax: +1 215 860 8192 info@heartsine.com

Europe/Rest of the World

HeartSine Technologies, Inc. HeartSine Technologies, Inc. 203 Airport Road West Belfast, Northern Ireland BT3 9ED Tel: +44 28 9093 9400 Fax: +44 28 9093 9401 info@heartsine.com

The products described in this brochure all meet the applicable European Medical Directive requirements.

0120

© 2014 HeartSine Technologies, Inc. All rights reserved. H009-036-203-0



www.heartsine.com

